CURAPAX COLD®

Patented

Version 01

Made in Denmark
CURAPAX COLD • Skeltoftevej 73 • DK-2800 Kgs Lyngby
curapax.dk

NO

CURAPAX COLD - Art. 5710329194063

Funksjon:

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CURAPAX COLD - Art. 5710329194063

Function:

- An easy, quick and cooling aid for injuries – just add water
- Lightweight and with a comfortable textile cover
- Allergy and environmentally friendly
- PVC and latex free

Instruction for use:

1. Fill the bag with water
2. Turn the bag upside down to lock the valve. Remove excess water from the opening.
3. The icepack is ready for use.

Warning:

If the opening is not emptied completely, excess water might spill. Avoid long-term or repeated skin contact with the contents of the ice pack. In case of contact, wash with soap and water. Seek medical help if irritation occurs. If you get the substance in your eyes, rinse immediately with large amounts of water. Seek medical help if irritations occur. Do not ingest the contents, but in case you do, contact a doctor.

An icepack might cause damage to the skin and the underlying tissue. For diabetics and people with circulatory diseases, cold therapy should only be used after doctor's recommendations. Be aware that for children, elderly and pregnant women, the skin might be especially sensitive, and supervision is advised. Be especially cautious when treating people with cutaneous insensitivity. The ice pack is only for external use. If the bag is damaged, do not use and discard. Always follow doctor's orders, avoid using cold therapy for more than 20 minutes at a time. ECO ICE PACK must not be used on open wounds and on damaged skin. Unintended use may cause damage. Do not use ECO ICE PACK with other products such as (but not limited to) sports cream, cooling balm, skin lotion.

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